

Instructions for Use

Dr. Reiß Dental First Aid

Dear Patient,

Please read the following instructions carefully !

The instructions contain important information that have to be observed during the treatment with “Dental First Aid”.

Please store the instructions with the set for the duration of product use and observe any additional information (if enclosed) for the individual components.

- ✓ The following treatments are for emergency use only, you have to consult a dentist immediately
- ✓ For further information regarding warnings and side effects please ask your dentist or pharmacist
- ✓ These instructions for use will be updated regularly
- ✓ Do not use after the expiration date
- ✓ Not for application by children (adult use only)
- ✓ Do not eat, do not apply to skin
- ✓ Do not store in direct sunlight
- ✓ Do not smoke during treatment and for 2 hours after the treatment
- ✓ Do not use fluoride compound during pregnancy

Content of the set:

	Content	Product Name
1	Temporary filling material for seals	Cavit®
2	Temporary Cement for crowns and bridges	TEMPBOND® unidose
3	Fluoride compound	UltraEZ®
4	Dental cotton rolls	DC Top Watterollen
5	Mixing pad	DC Anmischblock
6	Probe, spatula, dental mirror, tweezers	INSTRU KIT
7	Rescuebox for tooth accidents	SOS Zahnbox

Range for Indications:

Temporary repair of lost crowns and inlays

Temporary repair of lost telescope crowns and bridges

Temporary replacement of lost fillings

Temporary cavity coverage

Temporary storage of knocked out teeth and broken pieces of teeth

Haemostasis in case of injuries in the mouth

Treating of hypersensitive teeth

Notes for individual components (if further information is enclosed):

1. Cavit® (temporary filling material):
Due to sensitivity to moisture, close the tube immediately after removing the required material. Use open tubes as soon as possible. Do not use after the expiration date.
2. TEMPBOND® unidose (Temporary repair of lost crowns and bridges):
May cause irritation to eyes, respiratory system and skin. Product contains Eugenol; allergic reactions to Eugenol can occasionally occur in patients. Keep cool, application at room temperature.
3. UltraEZ® (Fluoride compound):
Do not apply with known allergy to fluoride supplements
4. SOS Zahnbox (Rescuebox for tooth accidents)
If the ready-to-use solution turns milky or changes color from light red to yellow, the liquid is contaminated and must not be used. This liquid was produced to be used only once.

Instructions:

1. Temporary repair of lost crowns and inlays and temporary repair of lost telescope crowns and bridges:

- Using the probe (sharp pointed instrument in the sealed pouch), remove the cement remnants from the crown and the tooth
- Place the crown onto the tooth to check fit
- Keep the tooth dry with the cotton roll (use tweezers to put cotton in mouth)
- Using equal parts, mix the two components of the TEMPBOND® unidose cement with the spatula (flat instrument at the end of the mirror) on the mixing block and apply to the crown
- Put the crown into mouth and press on the tooth
- Bite and wait 3 minutes
- Open mouth and remove any excess cement with the probe and the mirror
- Do not eat for 30 minutes

2. Temporary replacing of fillings dropped out/ broken teeth/ temporary closing of caries holes:

- Keep the tooth dry with the cotton roll (use tweezers to put cotton roll in mouth)
- Take a sufficient quantity of the temporary filling material (Cavit®) with the spatula from the tube and apply directly to the affected tooth
- Bite and grind teeth in all directions for 2 minutes, then wait 15 minutes
- Close tube after use and protect content against moisture
- Open mouth and remove any excess cement with the probe and the mirror
- Close mouth tightly in order to optimize the height (bite and grind)
- Avoid chewing on the treated areas for about 2 hours

3. Temporary storage of knocked out tooth and broken pieces of tooth:

- Handle the knocked out tooth or the tooth pieces only at the tooth crown, never touch the tooth root or site of fracture
- Do not clean tooth, not even if it is dirty
- The tooth / the pieces can be stored at room temperature in the SOS Zahnbox up to 24 hours in the ready-to-use solution
- Open the SOS Zahnbox and put tooth or the pieces of a tooth into it. Close it firmly. Immediately go to a dentist or dental clinic
- Bleeding in the mouth can be stopped by biting on a cotton roll or two (for 3-5 minutes).
If necessary, repeat the procedure

4. Treating of hypersensitive teeth:

- Keep the sensitive teeth dry with the cotton roll (use tweezers to put cotton roll in mouth)
- Take a sufficient quantity of UltraEZ® from the syringe on the spatula and apply directly to the sensitive place. Recommended treatment times can range from 15 minutes to 1 hour. During this time the cotton roll should stay in mouth
- Then clean your teeth with a toothbrush and rinse twice
- Close syringe after use and protect content against moisture
- Observe UltraEZ® plastic bag printing

5. Haemostasis in case of injuries in the mouth:

Bleeding in the mouth can be stopped by biting on a cotton roll or two (for 3-5 minutes).
If necessary, repeat the procedure

Symbole



CONSULT INSTRUCTIONS FOR USE



BATCH CODE



USE BY



MANUFACTURER



© R+R Ideenschmiede GmbH Leimitzer Straße 77
U-Ident-Nr. DE 264436483 HR B 4459

95028 Hof – Germany
Fax: +49(0)9281-470448